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
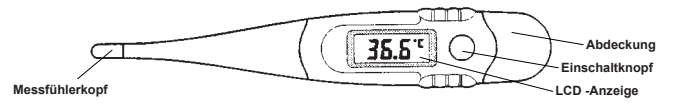
**INSTRUCTION FOR USE** Please read carefully before using.

**PARTS DESCRIPTION**
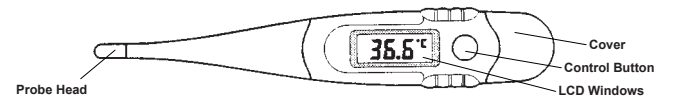
<b>SPECIFICATIONS</b>	
Range:	32.0 <span> </span> °C to 43.9 <span> </span> °C Below 32.0 <span> </span> °C display Lo°C Above 43.9 <span> </span> °C display Hi°C
Intended use:	Measurement of body temperature
Resolution:	0.1 <span> </span> °C
Accuracy:	±0.1 <span> </span> °C 34.0 <span> </span> °C to 42.0 <span> </span> °C
Display:	Liquid crystal display 3 1/2 digits
Battery (included):	Micro Alkaline 192 or LR41 or AG3; 1.5 V
Power consumption:	0.15 milliwatts in measurement mode
Battery life:	More than 400 continuous measurements
IP classification:	IP 22 (effective against solid foreign objects with a finger; protected against dripping water)
Dimensions:	Approx. 133.5 mm x 22 mm x 11.5 mm
Weight:	Approx. 17 grams including battery
Beeper:	Approx. 5 secs. sound signal when no relevant increase in temperature is detected
Memory:	Built-in memory enabling display of the previously measured value
Working Conditions:	Temperature: 10 <span> </span> °C to 45 <span> </span> °C, Humidity: 15 <span> </span> % to 95 <span> </span> % R.H. <div style="border: 1px solid black; padding: 2px;"> <p><span style="color:red">⚠</span> It is only possible to measure temperatures that are above the ambient temperature.</p> </div>
Storage Conditions:	Temperature: -25 <span> </span> °C to +55 <span> </span> °C, Humidity: 15 <span> </span> % to 95 <span> </span> % R.H.
Guarantee of Quality:	Conformity DIN EN 12470-3, Medical Thermometers part 3: Compact electrical thermometers with maximum device. Geratherm® is certified in accordance with Council Directive 93/42/EWG and EN ISO 13485 and is entitled to affix the CE-mark <b>CE0197</b> (Notified Body: TÜV Rheinland LGA Products GmbH).
Electromagnetic compatibility (EMC):	The present medical product is a device with extremely high degree of noise immunity. Therefore, the enclosure with the EMC tables was not printed out.

The present medical product is a device with extremely high degree of noise immunity. Therefore, the enclosure with the EMC tables was not printed out.

### HOW TO OPERATE

- Press down the control button to start the thermometer. Then, a full display 99.9 ensures that the LCD window is in good function.
- The next appearance Lo signals normal operation after internal safe testing. Otherwise, Err alerts its malfunction.
- Then, the ambient temperature or Lo displays on window with °C blinking. The thermometer is now ready for a temperature taking. If there is no temperature taking in the coming 3 minutes, the thermometer will be automatically shut-off.
- Clean or disinfect the probe before every measurement. Put the probe head under the tongue, under the arm or in the rectum; according to the methods provided (see Methods of Use).
- As soon as the thermometer does not register any further relevant increase in temperature, °C stops flashing and the maximum value measured up to that time is shown, accompanied by a series of audible signals.
- The thermometer continues measuring the temperature for a further minute after the audible signals begin, and in the time there may possibly be a further slight increase in the temperature shown. After that, the maximum temperature measured is recorded and stored; it remains visible on the LCD display for 9 minutes until the thermometer is automatically switched off.
- When the measured temperature is above or below Measuring range, the LCD window displays Hi or Lo.

### HOW TO RECALL THE LAST TEMPERATURE

Please keep pressing the control button about approximately 1.5 seconds. The last temperature will be recalled on the LCD window with "M" flag appeared at the right-bottom corner. For example 37.0.

### METHODS OF USE

⚠ It is only possible to measure temperatures that are above the ambient temperature.

**Axillary Measurement** (approximately 60 – 120 seconds)  
Dry the armpit with a towel. Place the probe head under the armpit. Close the arm firmly.  
In general, the axillary temperature is around 0.5 °C to 1.0 °C lower than oral temperature. Note that in order to achieve better axillary temperature measurement result, **a minimum measuring time of 3 to 5 minutes is recom-mended regardless of the beep sound**.

**Oral Measurement** (approximately 45 – 90 seconds)  
Place the probe head under the tongue. Assure a fixed and good contact. Close the mouth and breathe normally. A normal temperature by this method is usually considered between 36. 0 °C and 37.5 °C.

**Rectal Measurement** (approximately 30 – 60 seconds)  
Lubricate the probe head with Vaseline for an easier insertion. Then, softly enforce the probe head into the rectum approximately 2 to 3 cm in depth.  
In general, the rectal temperature is around 0.5 °C to 1.0 °C **higher** than oral temperature.

⚠ If you compare temperatures between two thermometers, it is possible to have two different readings even if both thermometers are within their accuracy tolerances. Differences in how you take each reading and the thermometer's tolerance range can account for nearly 0.1 °C variability.

### HOW TO REPLACE THE BATTERY

When the battery flag   appears and blinks, it means time to replace the battery. In this case,

- Turn the thermometer over, with the label side facing to the user. Pull out the cover as the following illustration.
- Remove the used battery by any pointed object but not sharp metal.
- Withdraw the battery.
- Insert a new 1.5 V button battery with the cathode side (-) up.
- Close the cover securely.

⚠ **Warning:** Do not dispose the battery in normal domestic waste! Out of reach of young children. Do not dispose of the battery in a fire. It may explode.

### CLEANING AND DISINFECTION

For household use, always clean the probe head before and after each use. Wash the tip with a solution of mild detergent and cool water. Disinfect the thermometer by wiping the sensor and lower stem with a cloth dipped in a household antiseptic solution such as rubbing alcohol (recommendation: 70% isopropyl alcohol). The thermometer is water resistant. Do not boil or clean in autoclave.

### PRECAUTIONS

- Use of this instrument is not intended as a substitute for consultation with your physician. Prolonged high fever requires medical attention especially for your children. Please contact your physician.
- Child's temperature shall be taken under adult's surveillance.
- Keep out of the reach of unsupervised children. Choking resulting from a child swallowing a small part that has become detached from the thermometer.
- Do not boil, bite, bend, drop or disassemble this thermometer.
- Do not attempt to disassemble the unit except to replace the battery.
- Avoid taking temperature in 30 minutes after physical exercises, bathing or dining.
- Do not store, use and transport the unit under direct sunlight, at a high temperature, or in high humidity or dust. Performance may be degraded.
- When the ambient temperature is greater than 35 °C, immerse the probe head into cold water for around 5 – 10 seconds before normal temperature taking.
- Stop using the thermometer if it operates erratically or if the display malfunctions.
- The possibility of allergic reactions and/ or skin irritation cannot be excluded completely.
- If the thermometer has been stored at below-freezing temperatures, allow the thermometer to warm naturally to room temperature before using.

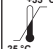
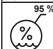
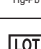
### DISPOSAL

Observe the applicable regulations when disposing of the thermometer and batteries. This thermometer must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmen-tally acceptable manner. Please remove the batteries before disposing of the thermometer. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

### WARRANTY

This thermometer is guaranteed for two years from the date of purchase against manufacturer's defect under normal use. If your unit does not function properly due to defective parts or assembly we will repair it free of charge. All parts are covered by this warranty excluding the battery. The warranty does not cover damage to your unit due to improper handling.

### SYMBOL INDEX

<span style="border: 1px solid black; padding: 1px;"></span> Follow instructions for use	<span style="border: 1px solid black; padding: 1px;"></span> Type BF applied part	<span style="border: 1px solid black; padding: 1px;"></span> Manufacturer
<span style="border: 1px solid black; padding: 1px;"></span> Store between -25 <span> </span> °C and +55 <span> </span> °C	<span style="border: 1px solid black; padding: 1px;"></span> Caution! Read the instruction manual.	<span style="border: 1px solid black; padding: 1px;"></span> The device must not be disposed of with household waste.
<span style="border: 1px solid black; padding: 1px;"></span> Store between 15 <span> </span> % and 95 <span> </span> % R.H.	<span style="border: 1px solid black; padding: 1px;"></span> Battery contains a small fraction of mercury and plumb must not be disposed of with household waste.	<span style="border: 1px solid black; padding: 1px;"></span>
<span style="border: 1px solid black; padding: 1px;"></span> Keep dry.	<span style="border: 1px solid black; padding: 1px;"></span> Batch code (mm/yyyy; month/year)	<span style="border: 1px solid black; padding: 1px;"></span> Geratherm Medical AG Fahrenheitstraße 1 D-98716 Geschwenda Germany www.geratherm.com

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