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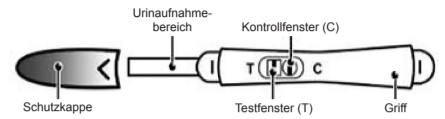
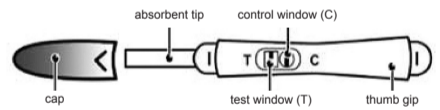
GB Geratherm® early detect

Early Detection Pregnancy Test - For self testing

PRINCIPLE

The Geratherm early detect pregnancy test is an early, lateral flow immunoassay in midstream format for the qualitative detection of human chorionic gonadotropin (hCG) in urine to aid in the detection of pregnancy. It can be used up to four days before the day your period is due, calculated 15 days after ovulation. The test utilizes a combination of antibodies including a monoclonal hCG antibody to selectively detect elevated levels of hCG. The assay is conducted by urinating on the absorbent tip or immersing the absorbent tip in urine, and obtaining the result from the colored lines.

PRECAUTIONS

Please read all the information in this package insert before performing the test. Do not use after the expiration date printed on the foil pouch. Store in a dry place at 2° - 30°C (or 36° - 86°F). Do not freeze. Do not use if pouch is torn or damaged. Please keep out of reach of children. For in vitro diagnostic use. For external use only. Open the foil pouch just before running the test.

KIT CONTENTS

REF 1: 1 aluminium foil pouch containing 1 test and 1 dessicant bag, 1 instructions for use

INSTRUCTIONS

Remove the midstream test from the foil pouch and familiarize yourself with the product. Remove the cap and place it over the thumb grip. Hold the test by the capped thumb grip with the exposed absorbent tip pointing downward directly into your urine stream for at least 10 seconds until it is thoroughly wet (See illustration below).

NOTE: Do not urinate on the test and control windows. If you prefer, you can urinate into a clean and dry container, then dip only the absorbent tip of the test into the urine for at least 10 seconds. After removing the test from your urine, immediately replace the cap over the absorbent tip, lay the test on a flat surface with the test and control windows facing upwards, and then begin timing. As the test begins to work, you may notice a light colored flow moving across the test and control windows. Read the result at 3 minutes. If no colored line appears, wait one minute longer. Some positive results may be observed in 1 minute or less depending on the concentration of hCG. Do not read the result after 10 minutes.

READING THE RESULTS

PREGNANT
Two distinct colored lines appear. One line should be in the Control Window (C) and another line should be in the Test Window (T). One line may be lighter than the other; they do not have to match. This means you are probably pregnant.

NOT PREGNANT
One colored line appears in the Control Window (C). No line appears in the Test Window (T). This means that you are probably not pregnant.

INVALID
The result is invalid if no colored line appears in the Control Window (C), even if a line appears in the Test Window (T). You should repeat the test with a new midstream test.

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CE0197

LIMITATIONS

There is the possibility that this test may produce false results. Consult your physician before making any medical decisions. Drugs which contain hCG can give a false positive result. Alcohol, oral contraceptives, painkillers, antibiotics or hormone therapies that do not contain hCG should not affect the test result. A number of medical conditions other than pregnancy, including, ovarian cyst or ectopic pregnancy (pregnancy outside the uterus) can cause elevated levels of hCG, which can result in a false positive result.

QUESTIONS AND ANSWERS

1. Q: How does the test work?
A: The Geratherm early detect pregnancy test detects a hormone in your urine that your body produces during pregnancy (hCG-human chorionic gonadotropin). The amount of pregnancy hormone increases as pregnancy progresses.
Respuesta: Quiere decir que su orina contiene hCG y que probablemente está embarazada. Vea su doctor para que le confirme que está embarazada y para discutir los pasos a seguir.
2. Q: How soon can I take the test
A: The test can be used up to 4 days before the day your period is due. You can perform the test anytime of the day, however, if you are pregnant, first morning urine contains the most pregnancy hormone.
3. Q: Do I have to test with first morning urine ?
A: Although you can test at any time of the day, your first morning urine is usually the most concentrated of the day and would have the most hCG in it.
4. Q: How accurate is the test?
A: A clinical evaluation was conducted comparing the results obtained using the early pregnancy test to another commercially available urine membrane hCG test. The consumer clinical trial included 608 urine specimens: both assays identified 231 positive and 377 negative results. Results have shown a test reliability of more than 99 % for pregnancy diagnosis on the day the period is due, compared to the second urine hCG test.
5. Q: How sensitive is the test?
A: The Geratherm early detect pregnancy test detects hCG in urine at a concentration of 10 IU/L or greater. The test has been standardized to the W.H.O. Third International Standard. The addition of LH (300 IU/L), FSH (1000 IU/L), and TSH (1,000 µIU/mL) to negative (0 IU/L) and positive (10 IU/L) specimens showed no cross-reactivity.
6. Q: What should I do if the result shows that I am pregnant?
A: It means that your urine contains hCG and you are probably pregnant. See your doctor to confirm that you are pregnant and to discuss the steps you should take.
7. Q: How do I know that the test was run properly?
A: The appearance of a colored line in the control window (C) tells you that you followed the test procedure properly and the proper amount of urine was absorbed.
8. Q: What should I do if the result shows that I am not pregnant ?
A: It means that no hCG has been detected in your urine and probably you are not pregnant. If you do not start your period within a week of its due date, repeat the test with a new midstream test. If you receive the same result after repeating the test and you still do not get your period, you should see your doctor.

You are encouraged to take the following steps to increase your chances for a healthy pregnancy and a healthy baby. The Geratherm early detect test can be used up to four days before the day your period is due, calculated 15 days after ovulation. You can begin better prenatal care as soon as you learn of your pregnancy. If you get a positive result, it is advisable to visit your doctor immediately to begin your prenatal care. Maintain a well-balanced diet, stop smoking, and reduce your intake of alcohol.

INDEX OF SYMBOL

	Attention, see instructions for use		Test per kit		Manufacturer
	For in vitro diagnostic use only		Use by		Do not reuse
	Store between 2 °C - 30 °C		Lot number		Catalog #

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Geratherm Medical AG